



Operating Manual

Resting and Stress ECG


1 Safety

2 Hardware

3 Software

4 Hygiene

Part 4: Hygiene and disinfection for custo cardio 100/110/130/300

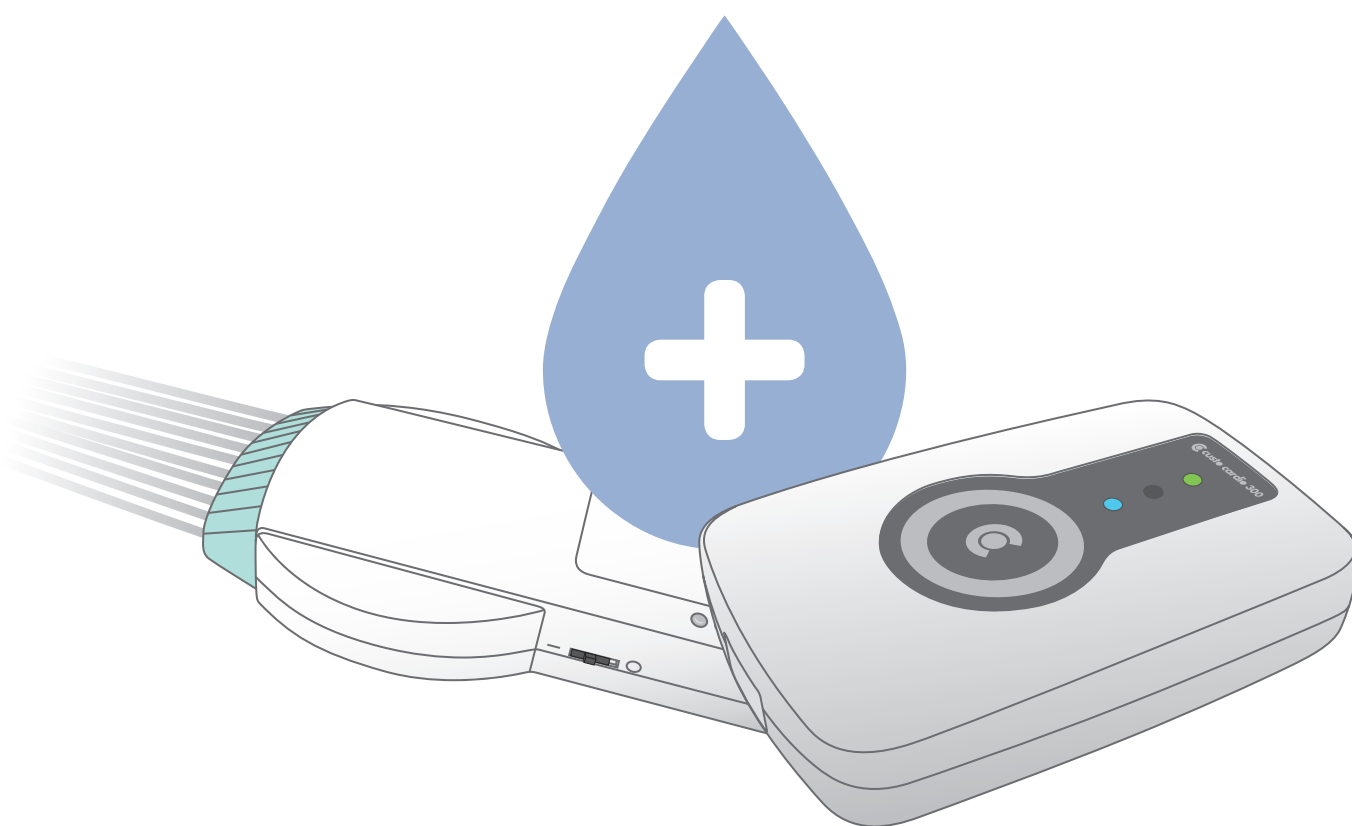

Operating characteristics:

for custo cardio 1xx/300
with custo diagnostic

MHY 0001 – DK 1702
Version 001 – 22/06/2017

CE 0123

 **custo-med**
EXCELLENCE IN DIAGNOSTICS



© 2017 custo med GmbH

This Operating Manual may not be copied in its entirety or in part, duplicated in any form or by any means or translated into another language without the prior written consent of custo med GmbH.

The manufacturer reserves the right to change the information in this Operating Manual without prior notice. The current version can be downloaded from our website: www.customed.de.

CAUTION:

This Operating Manual is part of a modular system consisting of four parts. All four parts must be downloaded from the Internet or from a CD to ensure the Operating Manual is complete.



Operating Manual

Resting and Stress ECG

1 Safety

2 Hardware

3 Software

4 Hygiene

Part 4: Hygiene and disinfection for custo cardio 100/110/130/300

Table of contents

4.1	Important information.....	4
4.2	Disinfection of custo cardio 100/110/130/300.....	4
4.3	Disinfection of patient cable	4
4.4	Disinfection of carrying case and belt.....	4
4.5	Approved cleaning agents and disinfectants.....	5
4.6	Disposal of contaminated consumables.....	5
	Cleaning overview at a glance	6

4.1 Important information

Use only cleaning agents and disinfectants that have been approved by custo med. Unsuitable agents may damage the device.

The devices must never be immersed in any liquid or cleaned with too much liquid. Cleaning agents and disinfectants must not be sprayed directly on or into the device. Moisture must not get into the inside of the device (e.g. via interface contacts).

The contacts must not be soiled or damaged.

Clean and disinfect the devices after each patient. Make sure that the exterior of the devices is always aesthetic and clean.

The device must not be connected to a power source during cleaning and disinfection.

4.2 Disinfection of custo cardio 100/110/130/300

- Reprocessing type: Wipe disinfection

4.3 Disinfection of patient cable

- Reprocessing type: Wipe disinfection

4.4 Disinfection of carrying case and belt

- Reprocessing type: washing with a disinfectant in a washing net

4.5 Approved cleaning agents and disinfectants

Wipe disinfection:

Meliseptol® Wipes sensitive (B.Braun)

Meliseptol® Foam pure (B.Braun) – (use with a soft, lint-free cloth)

➤ Note: Please follow the manufacturer's instructions

Washing with a disinfectant:

Eltra 40® (ECOLAB)

Eltra 40® Extra (ECOLAB)

➤ Note: Please follow the manufacturer's instructions



Machine wash at 40°C in a washing net (do not spin)



Do not treat with harsh chemicals or bleach



Do not iron



Do not tumble dry

➤ Please note: The approved disinfectants may be replaced by products from other manufacturers, provided these are equivalent with regard to disinfection and material compatibility. Please contact your hygiene & disinfection partner.

4.6 Disposal of contaminated consumables

Contaminated consumables such as adhesive electrodes (single-use items) are considered as waste with special requirements regarding collection and disposal from an infection prevention perspective. They must be disposed of in a safe and proper manner. Please observe the infection prevention legislation and the legal requirements for the disposal of contaminated consumables.



Cleaning overview at a glance

Product	Wipe disinfection	Washing and disinfection	Other
custo cardio 100/110/130/300	X		
custo cardio 200/400 and electrodes			S
custo screen 300/400/pediatric	X		
Blood pressure cuff			S
custo flash 500/510/510V/501/501L	X		
custo watch	X		
custo kybe	X		
custo guard	X		
custo belt electrode belt		X	
custo belt extender		X	
custo ECG cable guard 3 / cable adapter	X		
custo spiro mobile			S
Carrying cases, belts, neck straps		X	

S Special reprocessing required – further information is available in the corresponding operating manual

Manufacturer's contact details:

custo med GmbH
 Maria-Merian-Straße 6
 85521 Ottobrunn
 Germany

Phone: +49 (0) 89 710 98 - 00
 Fax: +49 (0) 89 710 98 - 10
 E-mail: info@customed.de
 Internet: www.customed.de

