



Operating Manual

Holter ECG


1 Safety

2 Hardware

3 Software

4 Hygiene

Part 4: Hygiene and disinfection for custo flash 500/510/510V/501/501L

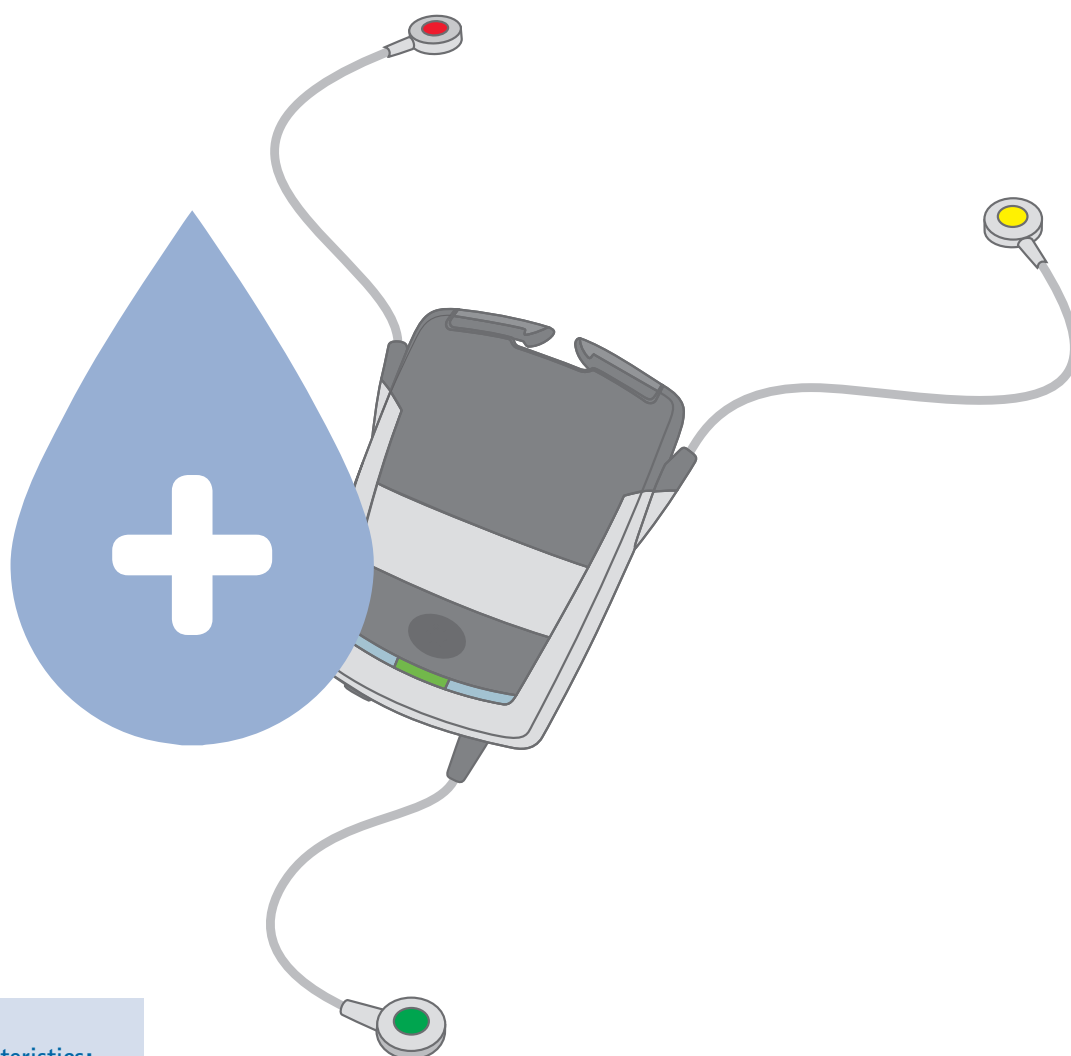

Operating characteristics:

for custo flash 5xxx
with custo diagnostic

MHY 0005 – DK 1706
Version 001 – 16/08/2017

CE 0123

 **custo·med**
EXCELLENCE IN DIAGNOSTICS



© 2017 custo med GmbH

This Operating Manual may not be copied in its entirety or in part, duplicated in any form or by any means or translated into another language without the prior written consent of custo med GmbH.

The manufacturer reserves the right to change the information in this Operating Manual without prior notice. The current version can be downloaded from our website: www.customed.de.

CAUTION:

This Operating Manual is part of a modular system consisting of four parts. All four parts must be downloaded from the Internet or from a CD to ensure the Operating Manual is complete.



Operating Manual

Holter ECG

1 Safety

2 Hardware

3 Software

4 Hygiene

Part 4: Hygiene and disinfection for custo flash 500/510/510V/501/501L

Table of contents

4.1	Important notes.....	4
4.2	Disinfection of the custo flash 500/510/510V/501/501L.....	4
4.3	Disinfection of the carrying case and the belt.....	4
4.3	Cleaning of the rechargeable battery charging cradle.....	4
4.5	Approved cleaning agents and disinfectants.....	5
4.6	Disposal of contaminated consumables.....	5
	Cleaning overview at a glance.....	6

4.1 Important notes

Use only cleaning agents and disinfectants that have been approved by custo med. Unsuitable agents could damage the device.

The devices must never be immersed in liquid or cleaned with too much liquid. Cleaning agents and disinfectants must not be sprayed directly on or into the device. Make sure that no moisture gets inside the device (e.g. via interface contacts).

Contacts must not become soiled or damaged.

Clean and disinfect the devices after each change of patient. Make sure that the exterior of the devices is always aesthetic and clean.

The device must not be connected to a power source during cleaning and disinfection.

4.2 Disinfection of the custo flash 500/510/510V/501/501L

- Reprocessing type: Wipe disinfection

4.3 Disinfection of the carrying case and the belt

- Reprocessing type: Washing with a disinfectant in a washing net

4.4 Cleaning of the rechargeable battery charging cradle

- Reprocessing type: Wipe down the dry device with a soft, lint-free cloth.

4.5 Approved cleaning agents and disinfectants

Wipe disinfection:

Meliseptol® Wipes sensitive (B.Braun)

Meliseptol® Foam pure (B.Braun) – (use with a soft, lint-free cloth)

► Note: Please observe the manufacturer's instructions

Washing with a disinfectant:

Eltra 40® (ECOLAB)

Eltra 40® Extra (ECOLAB)

► Note: Please observe the manufacturer's instructions

- | | |
|---|--|
|  | Machine wash at 40°C in a washing net, (do not spin) |
|  | Do not treat with harsh chemicals or bleach |
|  | Do not iron |
|  | Do not tumble dry |
-

► Note: The approved disinfectants can be replaced by products from other manufacturers provided they are equivalent in terms of disinfection and material compatibility. For more information, contact your partner for hygiene and disinfection.

4.6 Disposal of contaminated consumables

Contaminated consumables, such as adhesive electrodes (single-use articles), are classified as waste whose collection and disposal is subject to special requirements in view of the prevention of infection. They must be disposed of in a safe and proper manner. Observe the Law on the Prevention and Control of Infectious Diseases and the legal requirements for the disposal of contaminated consumables.



Cleaning overview at a glance

Product	Wipe disinfection	Washing with a disinfectant	Miscellaneous
custo cardio 100/110/130/300	X		
custo cardio 200/400 and electrodes			S
custo screen 300/400/pediatric	X		
Blood pressure cuff			S
custo flash 500/510/510V/501/501L	X		
custo watch	X		
custo kybe	X		
custo guard	X		
custo belt electrode belt		X	
custo belt extender		X	
custo ECG cable guard 3 /cable adapter	X		
custo spiro mobile			S
Carrying cases, carrying belt, neck strap		X	

S Special reprocessing necessary – more information can be found in the respective Operating Manual

Manufacturer's contact details:

custo med GmbH
 Maria-Merian-Straße 6
 85521 Ottobrunn
 Germany

Phone: +49 (0) 89 710 98 - 00
 Fax: +49 (0) 89 710 98 - 10
 E-mail: info@customed.de
 Internet: www.customed.de

