



Operating Manual

# Holter


1 Safety

2 Hardware

3 Software

4 Hygiene

## Part 4: Hygiene and disinfection for custo guard holter and accessories

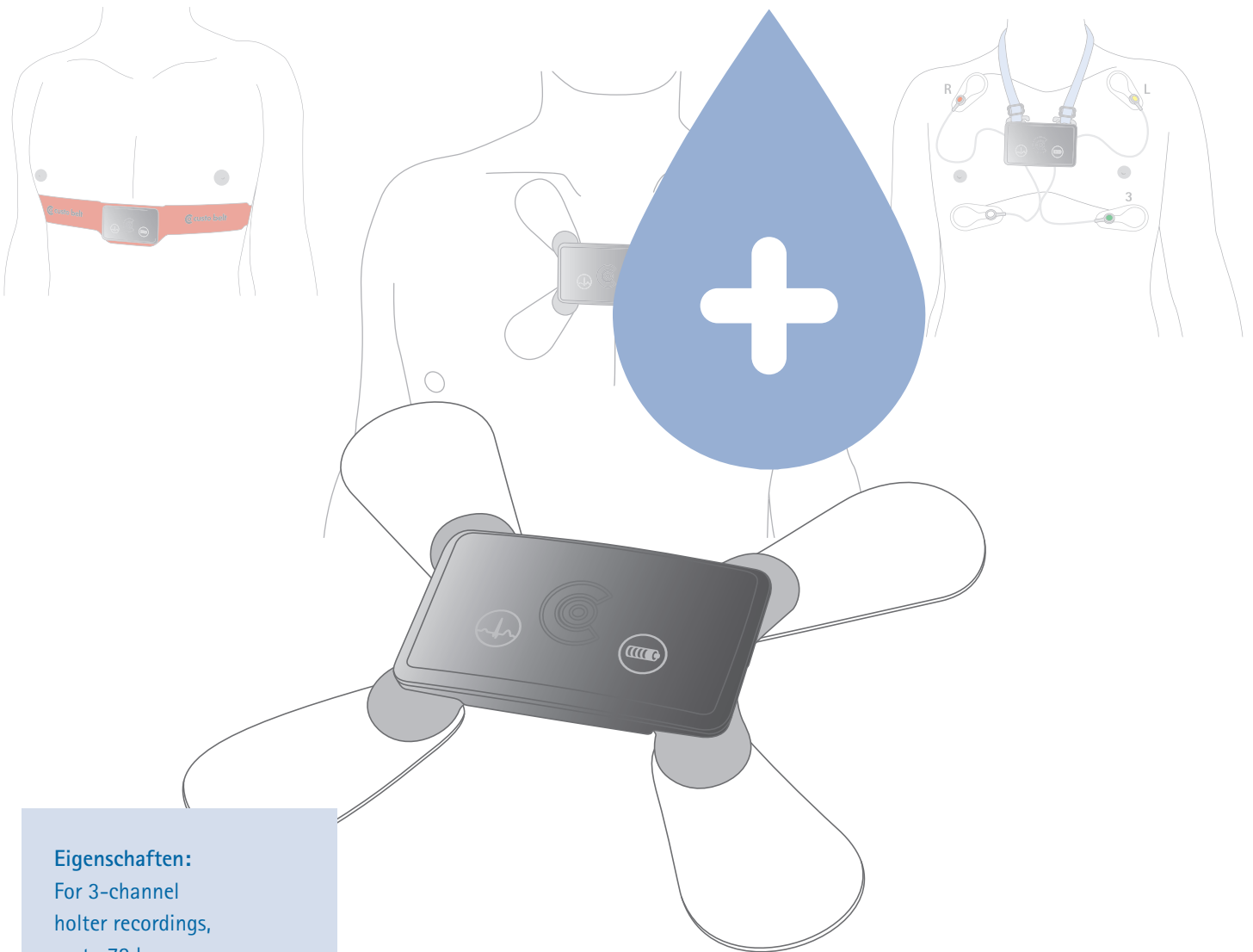

**Eigenschaften:**  
For 3-channel  
holter recordings,  
up to 72 hours

MHY 0010 – DK 1858  
Version 002 – 08/03/2021

CE 0123

**custo·med**  
EXCELLENCE IN DIAGNOSTICS



© 2021 custo med GmbH

This Operating Manual may not be copied in its entirety or in part, duplicated in any form or by any means or translated into another language without the prior written consent of custo med GmbH.

The manufacturer reserves the right to change the information in this Operating Manual without prior notice. The current version can be downloaded from our website: [www.customed.de](http://www.customed.de).

**CAUTION:**

**This Operating Manual is part of a modular system, consisting of four parts. All four parts must be downloaded from the Internet or from a CD to ensure the Operating Manual is complete.**



Operating Manual

# Holter

1 Safety

2 Hardware

3 Software

4 Hygiene

Part 4: Hygiene and disinfection for custo guard holter and accessories

## Table of contents

4.1	Important notes.....	4
4.2	Hygienic reprocessing custo med holter devices and accessories	
4.2.1	custo guard holter, adapter for custo wing electrodes and ECG cable guard 3/4.....	5
4.2.2	Neck strap (for ECG cable guard 3/4).....	5
4.2.3	custo guard base charging and communication unit.....	5
4.2.4	custo belt 1/3 and custo belt extender.....	5
4.3	Recommended cleaning agents and disinfectants.....	6
4.4	Disposal of contaminated consumables.....	6
	Cleaning overview at a glance.....	7

## 4.1 Important notes

---

Use only cleaning agents and disinfectants that have been approved by custo med. Unsuitable agents could damage the device.

---

The devices must never be immersed in liquid or cleaned with too much liquid. Cleaning agents and disinfectants must not be sprayed directly on or into the device.

Make sure that no moisture gets inside the device (e.g. via interface contacts).

---

Contacts must not become soiled or damaged.

---

Clean and disinfect the devices after each change of patient. Make sure that the exterior of the devices is always aesthetic and clean.

---

The device must not be connected to a power source during cleaning and disinfection.

---

## 4.2 Hygienic reprocessing custo med holter devices and accessories

### 4.2.1 custo guard holter, adapter for custo wing electrodes and ECG cable guard 3/4

- Reprocessing type: Wipe disinfection

custo guard holter must not be stored damp in the custo guard base charging and communication unit. Only store dry in the charging unit.

### 4.2.2 Neck strap (for ECG cable guard 3/4)

- Reprocessing type: disinfectant washing in the washing net

### 4.2.3 custo guard base charging and communication unit

- Reprocessing type: Wipe down the dry device with a soft, lint-free cloth.

### 4.2.4 custo belt 1/3 and custo belt extender

**ATTENTION:** custo belt must be washed before first use to remove any production residues and restore the initial condition of the medical device.

- Reprocessing type: disinfectant washing in the washing net
- Note: Lay flat to dry. Do not hang dry.  
Allow to dry completely before stretching or using it.  
Up to 100 washing cycles.

**ATTENTION:** After use on infectious patients, check whether the corresponding germs have been destroyed by the disinfectant washing process (refer to the manufacturer's information on the detergent). If this is not the case, the custo belt and the extender may no longer be used.

### 4.3 Recommended cleaning agents and disinfectants

#### Wipe disinfection:

- Meliseptol® Wipes sensitive (B.Braun)
- Meliseptol® Foam pure (B.Braun), use with a soft, lint-free cloth
- Please observe the manufacturer's instructions.

#### Washing with a disinfectant:

- Eltra 40® Extra (ECOLAB)
- Please observe the manufacturer's instructions.



Machine wash at 40°C in a washing net (do not spin)



Do not treat with harsh chemicals or bleach



Do not iron



Do not tumble dry

The recommended disinfectants can be replaced by products from other manufacturers provided they are equivalent in terms of disinfection and material compatibility. For more information, contact your partner for hygiene & disinfection.

### 4.4 Disposal of contaminated consumables

Contaminated consumables, such as adhesive electrodes (single-use articles), are classified as waste whose collection and disposal is subject to special requirements in view of the prevention of infection. They must be disposed of in a safe and proper manner. Observe the Law on the Prevention and Control of Infectious Diseases and the legal requirements for the disposal of contaminated consumables.



## Cleaning overview at a glance

Product	Wipe disinfection	Washing with a disinfectant	Miscellaneous
custo cardio 100/110/130/300	X		
custo cardio 200/400 Et electrodes			S
custo screen 300/310/400/pediatric	X		
Blood pressure cuff			S
custo flash 500/510/510V/501/501L	X		
custo watch	X		
custo kybe	X		
custo guard 1/3	X		
custo guard holter	X		
custo belt 1/3		X	
custo belt extender		X	
ECG cable guard 3/4, cable adapters	X		
custo spiro mobile			S
Carrying cases, carrying belts, neck straps		X	

S Special reprocessing is necessary – for more information refer to the respective Operating Manual

### Manufacturer's contact details:

custo med GmbH  
 Maria-Merian-Straße 6  
 85521 Ottobrunn  
 Germany

Phone: +49 (0) 89 710 98 - 00  
 Fax: +49 (0) 89 710 98 - 10  
 E-mail: [info@customed.de](mailto:info@customed.de)  
 Internet: [www.customed.de](http://www.customed.de)

