



Operating Manual

Cardiac Rehabilitation


1 Safety

2 Hardware

3 Software

4 Hygiene

Part 4: Hygiene and disinfection for custo guard 1/3 and accessories

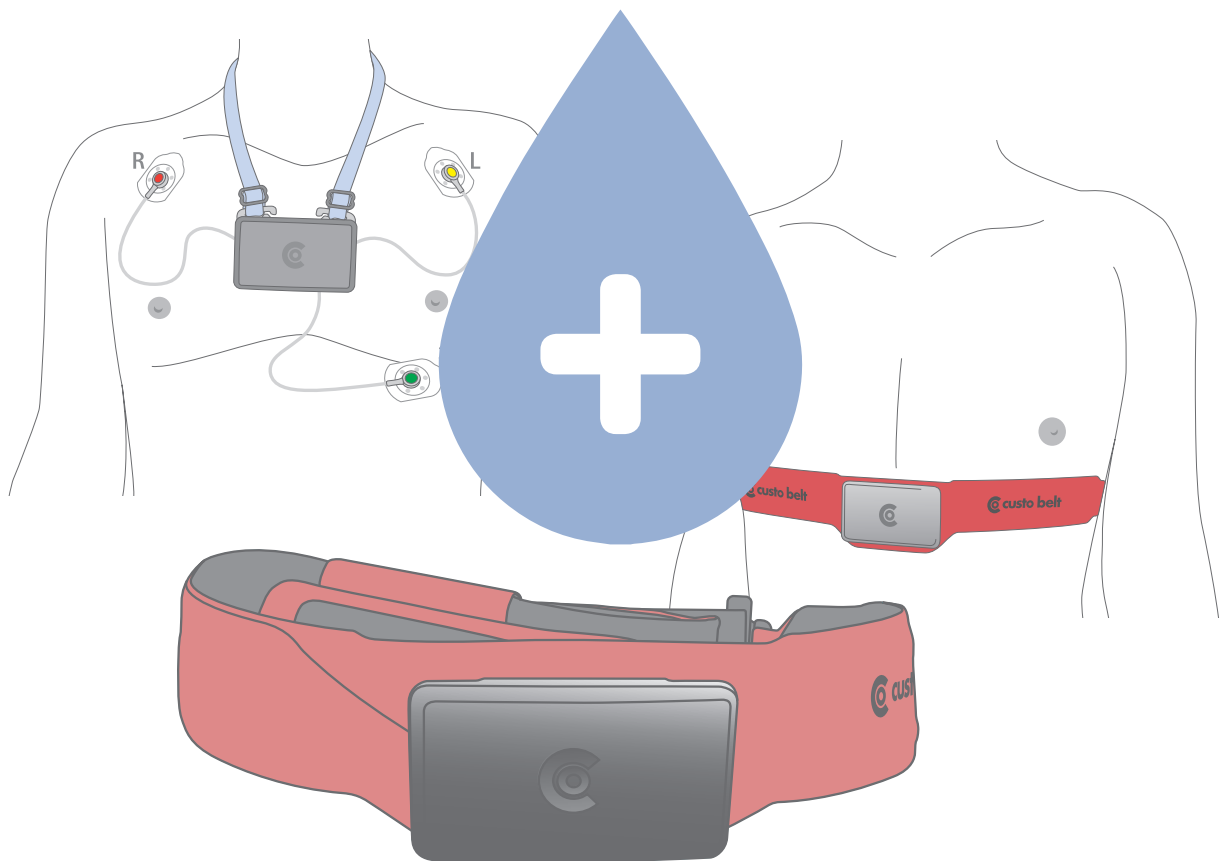

Operating characteristics:

1- or 3-channel
ECG transmitter with
electrode belt for wireless
patient monitoring
MHY 0008 – DK 1709
Version 002 – 21/08/2019

CE 0123

 **custo·med**
EXCELLENCE IN DIAGNOSTICS



© 2019 custo med GmbH

This Operating Manual may not be copied in its entirety or in part, duplicated in any form or by any means or translated into another language without the prior written consent of custo med GmbH.

The manufacturer reserves the right to change the information in this Operating Manual without prior notice. The current version can be downloaded from our website: www.customed.de.

CAUTION:

This Operating Manual is part of a modular system, consisting of four parts. All four parts must be downloaded from the Internet or from a CD to ensure the Operating Manual is complete.



Operating Manual

Cardiac Rehabilitation

1 Safety

2 Hardware

3 Software

4 Hygiene

Part 4: Hygiene and disinfection for custo guard 1/3 and accessories

Table of contents

4.1	Important notes.....	4
4.2	Disinfection of custo guard 1/3, cable adapter and ECG cable custo guard 3.....	4
4.3	Disinfection of custo belt and custo belt extender.....	4
4.4	Disinfection of custo guard carrying case and neck strap	5
4.5	Cleaning of custo charger (single and multi).....	5
4.6	Recommended cleaning agents and disinfectants.....	5
4.7	Disposal of contaminated consumables.....	5
	Cleaning overview at a glance	6

4.1 Important notes

Use only cleaning agents and disinfectants that have been approved by custo med. Unsuitable agents could damage the device.

The devices must never be immersed in liquid or cleaned with too much liquid. Cleaning agents and disinfectants must not be sprayed directly on or into the device.

Make sure that no moisture gets inside the device (e.g. via interface contacts).

Contacts must not become soiled or damaged.

Clean and disinfect the devices after each change of patient. Make sure that the exterior of the devices is always aesthetic and clean.

The device must not be connected to a power source during cleaning and disinfection.

4.2 Disinfection of custo guard 1/3, cable adapter and ECG cable custo guard 3

- Reprocessing type: Wipe disinfection
Do not store the custo guard when wet in the custo charger, ensure the device is dry before placing it in the charger.

4.3 Disinfection of custo belt and custo belt extender

ATTENTION: custo belt must be washed before first use to remove any production residues and restore the initial condition of the medical device.

- Reprocessing type: Washing with a disinfectant in a washing net
- Note: Lay flat to dry. Do not hang dry.
Allow to dry completely before stretching or using it.
Up to 100 washing cycles.

ATTENTION: After use on infectious patients, check whether the corresponding germs have been destroyed by the disinfectant washing process (refer to the manufacturer's information on the detergent). If this is not the case, the custo belt and the extender may no longer be used.

4.4 Disinfection of custo guard carrying case and neck strap

- Reprocessing type: Washing with a disinfectant in a washing net

4.5 Cleaning of custo charger (single and multi)

- Reprocessing type: Wipe down the dry device with a soft, lint-free cloth.

4.6 Recommended cleaning agents and disinfectants

Wipe disinfection:

- Meliseptol® Wipes sensitive (B.Braun)
- Meliseptol® Foam pure (B.Braun), use with a soft, lint-free cloth
- Please observe the manufacturer's instructions.

Washing with a disinfectant:

- Eltra 40® (ECOLAB)
- Eltra 40® Extra (ECOLAB)
- Please observe the manufacturer's instructions.



Machine wash at 40°C in a washing net, (do not spin)



Do not treat with harsh chemicals or bleach



Do not iron



Do not tumble dry

The recommended disinfectants can be replaced by products from other manufacturers provided they are equivalent in terms of disinfection and material compatibility. For more information, contact your partner for hygiene & disinfection.

4.7 Disposal of contaminated consumables

Contaminated consumables, such as adhesive electrodes (single-use articles), are classified as waste whose collection and disposal is subject to special requirements in view of the prevention of infection. They must be disposed of in a safe and proper manner. Observe the Law on the Prevention and Control of Infectious Diseases and the legal requirements for the disposal of contaminated consumables.



Cleaning overview at a glance

Product	Wipe disinfection	Washing with a disinfectant	Miscellaneous
custo cardio 100/110/130/300	X		
custo cardio 200/400 & electrodes			S
custo screen 300/400/pediatric	X		
Blood pressure cuff			S
custo flash 500/510/510V/501/501L	X		
custo watch	X		
custo kybe	X		
custo guard 1/3	X		
custo belt 1/3		X	
custo belt extender		X	
ECG cable custo guard 3 / cable adapter	X		
custo spiro mobile			S
Carrying cases, carrying belts, neck straps		X	

S Special reprocessing is necessary – for more information refer to the respective Operating Manual

Manufacturer's contact details:

custo med GmbH
 Maria-Merian-Straße 6
 85521 Ottobrunn
 Germany

Phone: +49 (0) 89 710 98 - 00
 Fax: +49 (0) 89 710 98 - 10
 E-mail: info@customed.de
 Internet: www.customed.de

