REF 1303

LiteAire® VALVED HOLDING CHAMBER

1 LiteAire

Manufacturer
THAYER MEDICAL CORPORATION
4575 S Palo Verde Rd, Ste 337
Tucson, AZ 85714-1961 USA
www.thayermedical.com

U.S. Patent No. 6,202,643; 6,550,473; 6,679,252 and International Patents.

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Made in USA

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician or other practitioner licensed to prescribe by law.

(01)00383480000495(11)151028(10)000000

2015-10

LOT 000000

EN Dual-Valved Holding Chamber

- Clean, Ready to Use • Easy Assembly • Printed with Soy Ink
- May be Flattened Between Uses • No Natural Latex

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
LITEAIRE LOGO and THAYER MEDICAL LOGO are registered trademarks of Thayer Medical Corporation

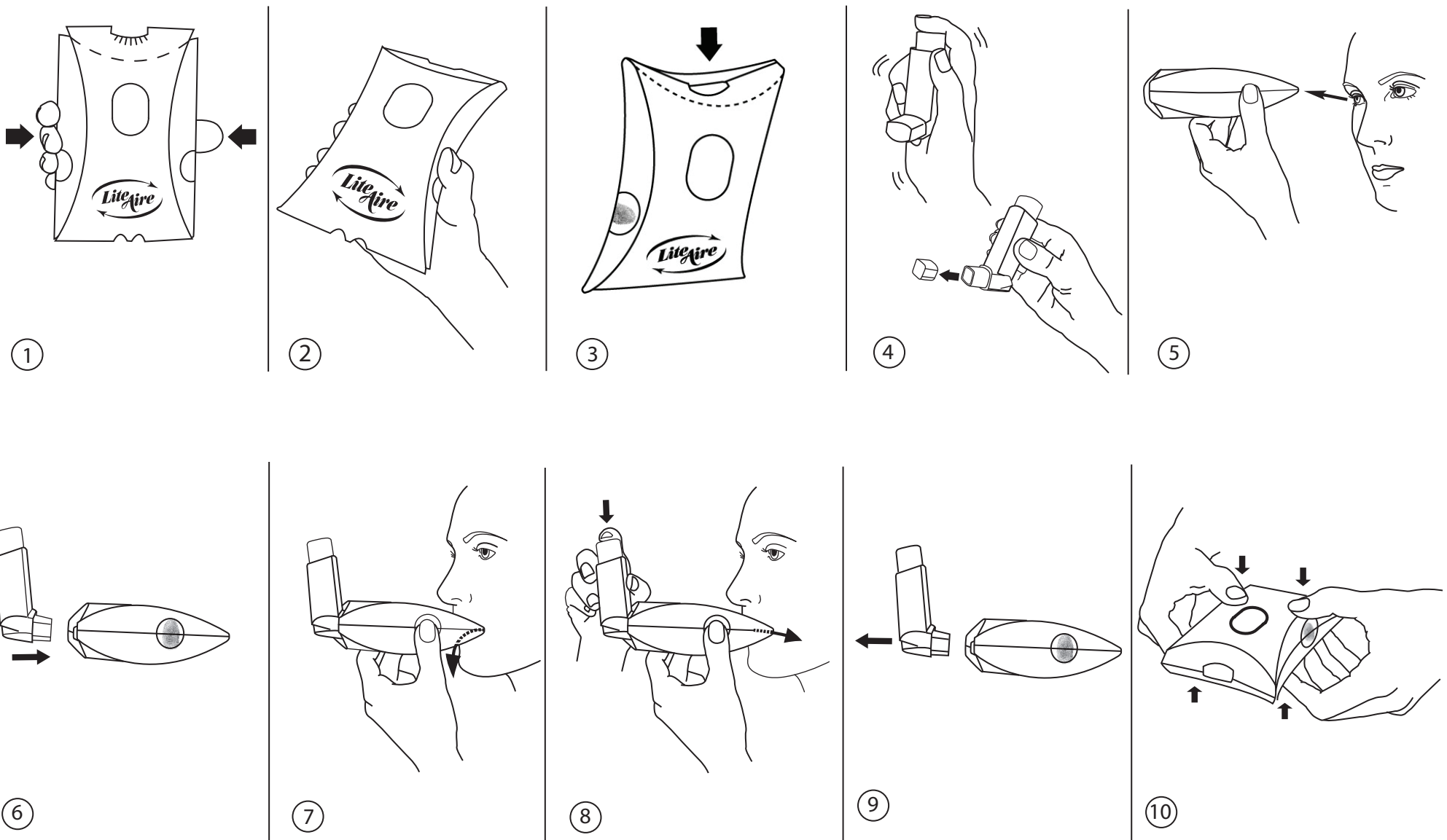© 2015 Thayer Medical Corporation

INSTRUCTIONS FOR USE (single patient use only)

EN ENGLISH

INTENDED USE

The LiteAir® is a portable, pop-up dual-valved holding chamber designed to be used with virtually all metered dose inhalers (MDIs). The purpose of the LiteAir® is to improve the delivery of the MDI medication to your lungs.

HOW TO ASSEMBLE YOUR LITEAIR®

1. Place your fingers in the middle of the LiteAir® along the long sides of the chamber. (Fig. 1)
2. Gently squeeze the device to make the LiteAir® pop up fully. The side panels will snap into place. (Fig. 2 and Fig. 3)

HOW TO USE YOUR LITEAIR® WITH YOUR MDI

1. Remove the cap from the inhaler mouthpiece. (Fig. 4)
2. Shake the inhaler prior to use as directed by the manufacturer. (Fig. 4)
3. Insert the inhaler canister mouthpiece firmly into the LiteAir® inhaler port. (Fig. 6)
4. Firmly grasp your LiteAir® with your thumb and fingers on either side at the point where you feel the paperboard inner wall. Gently squeeze the side panels to ensure the most effective treatment. (Fig. 7)
5. Close your lips firmly around the small holes on the mouthpiece of the LiteAir®.
6. Exhale fully into the mouthpiece (Fig. 7). As you begin to inhale, press down on the metal canister to spray one puff of medication. Continue inhaling slowly and deeply. Hold your breath for 5 to 10 seconds. (Fig. 8)
7. If your physician has directed you to take more than one puff, repeat step 6.
8. Remove the inhaler and flatten LiteAir® between uses. (Fig. 9 and Fig. 10)

FOR BEST RESULTS

1. Hold side panels near the mouthpiece where you feel the paperboard inner wall.
2. For maximum results, start inhaling at the same time that you squeeze the MDI canister.
3. Inhale as slowly and deeply as possible through your mouth, to fill your lungs with the medication. Be sure to hold your breath for 5 to 10 seconds.
4. Each LiteAir® can be used for multiple puffs. We recommend that you replace your LiteAir® weekly or sooner if it is crushed, soggy, contaminated or discolored.

WARNING

Always look inside the mouthpiece and chamber before you use the LiteAir® to be sure there are no small objects that could be inhaled (such as coins, lint, buttons, etc.).

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