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INSTRUCTIONS FOR USE FOR POSITIONING AID FOR FIBEROPTIC PAD SYSTEMS BILISWADDLER

WARNING: The instructions for use of the basic machine must be strictly observed!

WARNING: For a full understanding of the performance characteristics of this medical device, the user should carefully read this manual before use of the medical device.

DEFINITIONS

WARNING: A WARNING statement provides important information about a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION: A CAUTION statement provides important information about a potentially hazardous situation, which, if not avoided, may result in a minor or moderate injury to the user or patient or in a damage to the equipment or other property.

NOTE: A NOTE provides additional information intended to avoid inconvenience during operation of the medical device.

INTENDED USE

Positioning aid for phototherapy with fiberoptic pads for added warmth and comfort. During the therapy session the baby can be breast fed or held without impairing the bonding between the infant and its parents.

PREPARATION AND OPERATION

WARNING: Inspect for defects and damage before using positioning aid BiliSwaddler. Use only undamaged products, otherwise filling material may escape. Danger of suffocation! If the product is damaged dispose in accordance with local regulations.

WARNING: Ensure that breathing is not obstructed by positioning aid BiliSwaddler. Danger of suffocation!

WARNING: Use only new or cleaned and dried parts. Risk of cross infection! Contaminated positioning aids BiliSwaddler should be replaced and cleaned.

CAUTION: When not in use, store positioning aids BiliSwaddler so that they are protected against contamination and soiling.

CAUTION: Please use appropriate eye-protection during photo therapy sessions.

NOTE: We recommend to use translucent disposable covers for fiberoptic light pads or mattresses.

CLEANING AND DISPOSAL

CAUTION: Even reusable accessories (e.g. after processing) have a limited life time. A number of factors may increase wear and reduce product life time considerably. When external signs of wear become apparent, (such as cracks, discoloration, peeling, etc.) these parts should be replaced.

Following use, the product must be disposed of in accordance with the local hospital, hygiene and waste disposal regulations. Cleaning should be performed in accordance with the washing labels using a tested and recognized washing disinfection process, preferably in a cleaning/disinfecting machine. Velcro fasteners should be closed before washing!

CAUTION: Do not feed positioning aid BiliSwaddler through a wringer or mangle!

wash at 60°C / 140°F

only oxygen bleaching / no chlorine bleach!

rotary drier

Do not iron

do not dry clean

AMBIENT CONDITIONS

Temperature during storage 0 to 50°C (32 to 122°F)

CLASSIFICATION

According to EC Directive 93/42/EEC

Appendix IX

Class I

UMDNS-Code*

Positioning Aid for Fiberoptic Pad Systems

11-093

* Universal Medical Device Nomenclature System

TECHNICAL DATA AND ORDER LIST

Part.No.	Designation	Material	Phototherapy panel systems	Dimensions
20-00-100	Positioning Aid for fiberoptic light pads BiliSwaddler	Fabric: 80% CO, 20% PES, Trevira bioactive Filling: 100% PES	GE Healthcare BiliSoft, Natus neoBLUE blanket, Atom Medical Bili-Therapy	onesize (26 cm x 50 cm)

INSTRUCTIONS FOR USE (PAGE 3)

- 1 Feed the phototherapy cable through the bottom opening of the BiliSwaddler.
- 2 Place the baby on the mattress/panel in the pouch.
- 3 Fix the right side flap on the left side below the phototherapy mattress/panel.
- 4 Wrap the baby with the left side flap.
- 5 Fix the side flap on the reverse side of the BiliSwaddler by using the two velcro pads.
- 6 Check and adjust the fit of the velcro pads.
- 7 Please use appropriate eye-protection during photo therapy sessions.
- 8 BiliSwaddler can be used in a baby bed or in the arms of a caregiver (kangaroo care).

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